



# Genvinset<sup>®</sup>

## HLA Celiac & HLA Celiac Plus



EUROPEAN UNION  
European Regional Development Fund  
A WAY TO MAKE EUROPE

### Molecular determination of HLA-DQB1\*02, DQB1\*03:02, DQA1\*05 and DQA1\*03 alleles

Kits for detecting the HLA-DQB1\*02, DQB1\*03:02,  
DQA1\*05 and DQA1\*03 alleles by Real-Time PCR  
using TaqMan<sup>®</sup> probes technology

## About Genvinset<sup>®</sup> HLA Celiac & Celiachia Plus Real Time

Code NLM AA1728/24 - AA1728/48  
Code NLM AA1542/24 - AA1542/48

Susceptibility to gluten sensitivity is, in part, genetically determined. The strong predisposition is associated with HLA-DQ alleles, encoding the  $\alpha$  and  $\beta$  chains of the Major Histocompatibility Complex (MHC) class II.

In most populations studied, 90-95% of patients carry the HLA-DQ2 heterodimer encoded by the alleles HLA-DQA1\*05 and HLA-DQB1\*02 in cis or trans position. The remaining patients (5-10%) usually carry a second heterodimer, HLA-DQ8, encoded by the alleles HLA-DQA1\*03:01 and HLA-DQB1\*03:02. It is estimated that only 0.5% of celiac patients do not carry HLA-DQ2 or HLA-DQ8 heterodimers.

## Genvinset® HLA Celiac - Intended use

Genvinset® HLA Celiac is a semi-automated *in vitro* diagnostic kit for the qualitative detection of the HLA-DQB1\*02, DQB1\*03:02 and DQA1\*05 alleles in genomic DNA extracted from whole blood, and the consequent determination of the DQ2 and DQ8 antigens, associated with celiac disease predisposition. The kit is able to determine the homozygosity or heterozygosity status for the DQB1\*02 alleles. The analysis is based on Real-Time PCR technology, using TaqMan® probes.

Patients who can benefit from this determination are those referred by a specialist. The results of this test should not be the only ones on which the therapeutic decision is based and should be used as an aid in the diagnosis together with results of other markers of the disease.

The intended user of the kit is technical personnel trained to carry out the protocol and the interpretation of results described in the Instructions for Use.

## Genvinset® HLA Celiac Plus - Intended use

Genvinset® HLA Celiac Plus is a semi-automated *in vitro* diagnostic kit for the qualitative detection of HLA-DQB1\*02, DQB1\*03:02, DQA1\*05 and DQA1\*03 alleles in genomic DNA extracted from whole blood, and the consequent determination of DQ2 and DQ8 antigens associated to celiac disease. The kit is able to determine the homozygosity or heterozygosity status for the DQB1\*02 alleles. The analysis is based on Real-Time PCR technology using TaqMan® probes.

Patients who can benefit from this determination are those referred by a specialist. The results of this test should not be the only ones on which the therapeutic decision is based and should be used as an aid in the diagnosis together with results of other markers of the disease.

The intended user of the kit is technical personnel trained to carry out the protocol and the interpretation of results described in the Instructions for Use.

## Highlights

- The kit resolves the zygosity of the HLA-DQB1\*02 allele.
- The Genvinset® Report Viewer software combines all reactions' results to determine the probable genotype and the associated risk to celiac disease.


## Workflow




## Limitations

- See detected alleles in the updated documents "HLA alleles detected\_GVS-DQ" and "HLA alleles detected\_GVS-DQP" at [www.bdrdiagnostics.com](http://www.bdrdiagnostics.com).
- Mutations or polymorphisms at annealing primer/probe sites are possible and may result in the lack of allele definition. Other technologies could be necessary to resolve the typing.
- Data and result interpretation should be revised by qualified personnel.
- This product is an auxiliary tool for the diagnosis of patients with suspected celiac disease. Use these results in conjunction with clinical data and results of other tests performed on the patient.

# Results HLA Celiac & HLA Celiac Plus

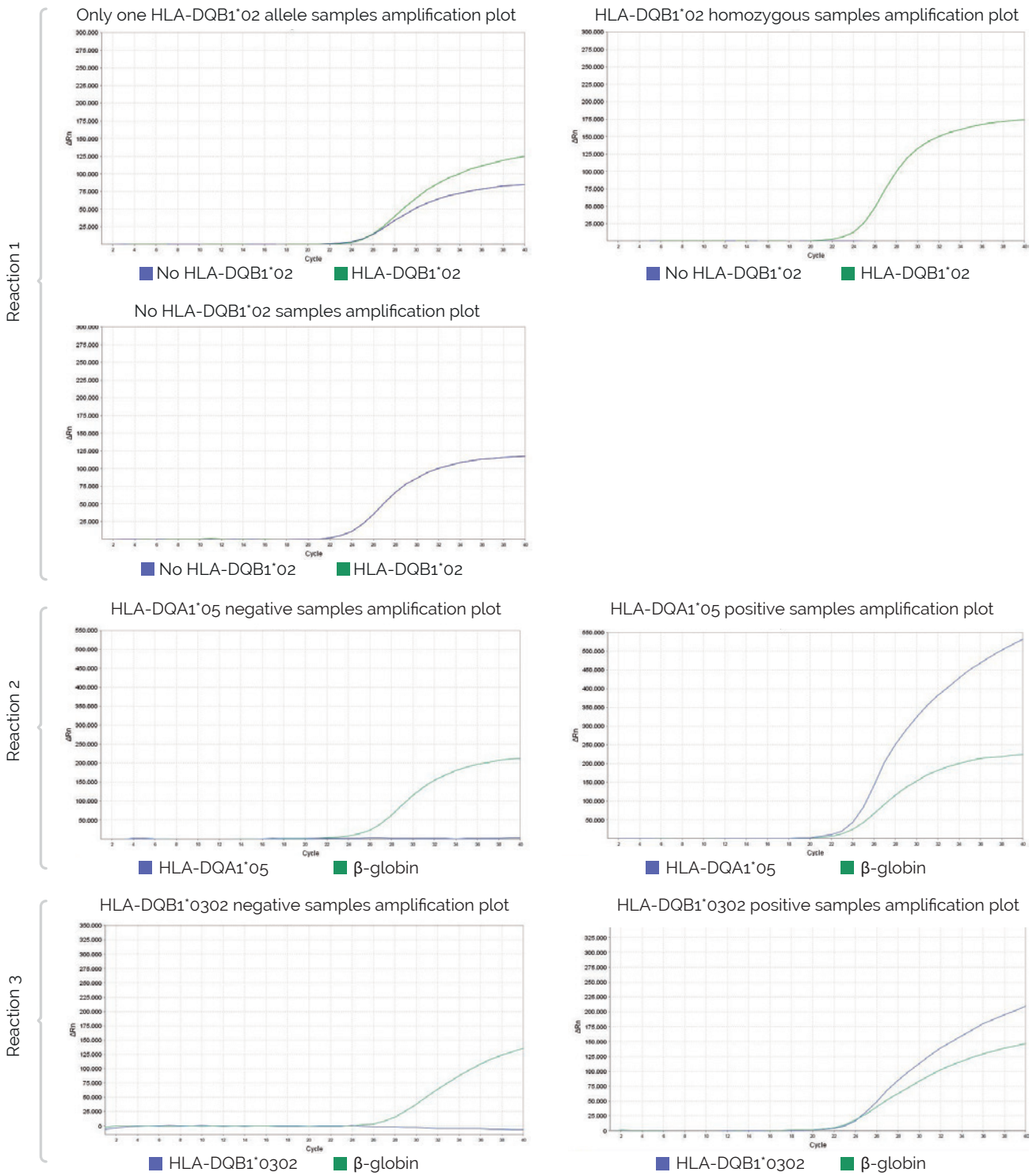

# Results HLA Celiac Plus

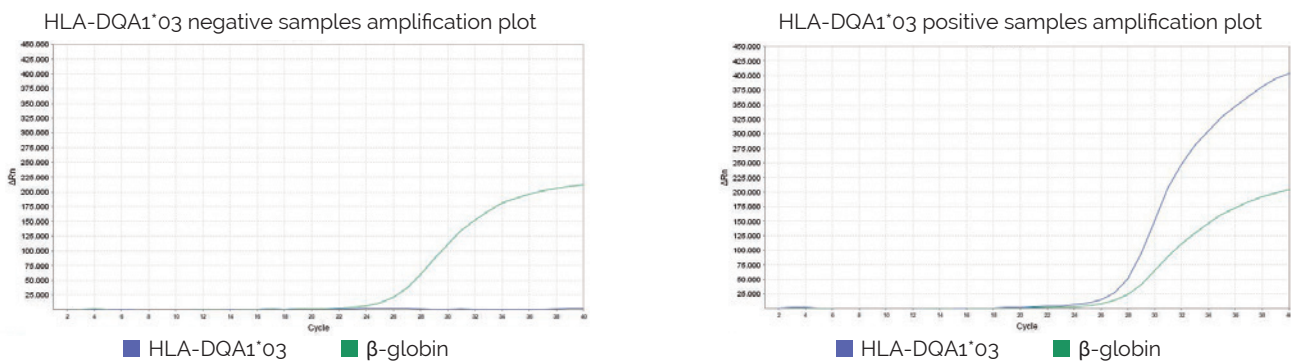

# Genvinset® HLA Celiac Product Information

24 tests - Code NLM AA1728/24  
48 tests - Code NLM AA1728/48

DESCRIPTION: Genvinset® HLA Celiac 24 test  
DESCRIPTION: Genvinset® HLA Celiac 48 test

CE-IVD certified

# Genvinset® HLA Celiachia Plus Real Time Product Information

24 tests - Code NLM AA1542/24  
48 tests - Code NLM AA1542/48

DESCRIPTION:  
Genvinset® HLA Celiachia Plus Real Time 24 test  
Genvinset® HLA Celiachia Plus Real Time 48 test

CE-IVD certified



**BDR**

**BLACKHILLS DIAGNOSTIC RESOURCES S.L.U.**

SRN: ES-MF-000001091 (Spain) · FIC GVS-DQ REV.04



**EXCLUSIVE DISTRIBUTOR  
Nuclear Laser Medicine srl**

Viale delle Industrie, 3 - 20049 Settala (Milan), ITALY  
Tel. 02 952451 - Fax 02 95245237-8

Partita Iva 08763060152  
segreteria@nlm.it - [www.nlm.it](http://www.nlm.it)